
**Bedienungsanleitung**  
**User manual**  
**Manual de instrucciones**  
**Mode d'emploi**

**TSE-A**

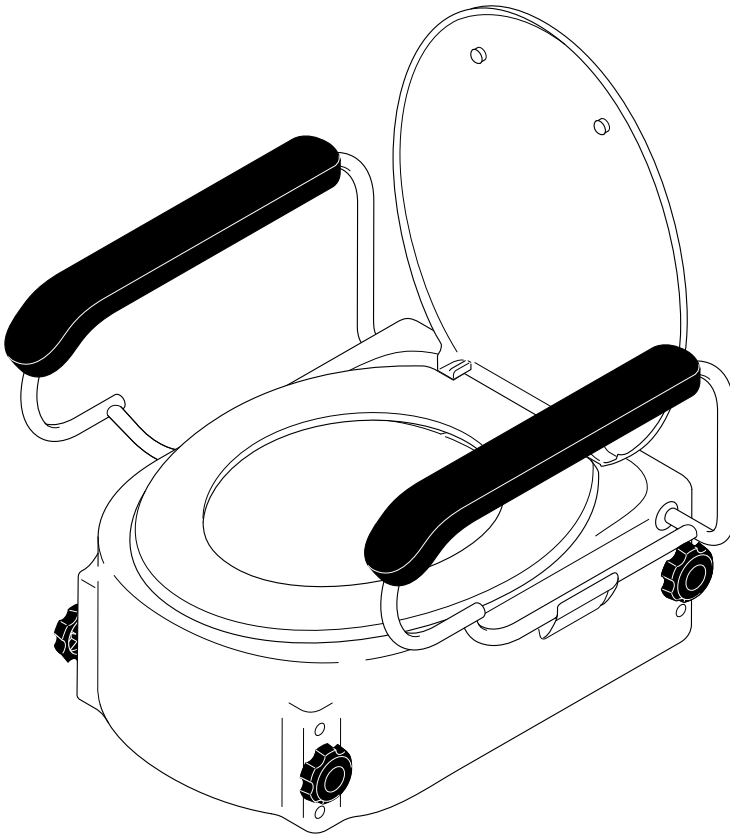DE

EN

ES

FR

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## Contents

<b>1 Introduction</b> .....	<b>14</b>	<b>4 Armrests assembly and/ or adjustment</b> .....	<b>19</b>
<b>2 Safety information</b> .....	<b>15</b>	<b>5 Adjusting the seat height</b> .....	<b>20</b>
2.1 Labels and symbols .....	15	<b>6 Mounting the toilet seat raiser</b> .....	<b>21</b>
2.2 Intended use .....	15	<b>7 Usage and care</b> .....	<b>22</b>
2.3 Indications.....	15	<b>8 Maintenance and repair</b> .....	<b>23</b>
2.4 Contraindications.....	15	<b>9 Transfer/Re-use</b> .....	<b>23</b>
2.5 General safety information.....	16	<b>10 Storage and shipping</b> .....	<b>23</b>
2.6 Special safety information .....	16	<b>11 Disposal</b> .....	<b>23</b>
2.7 Service life .....	17	<b>12 Warranty</b> .....	<b>24</b>
2.8 Declaration of compliance.....	17	12.1 Warranty certificate .....	24
<b>3 Components/Overview</b> .....	<b>17</b>	<b>Explanation of symbols</b> .....	<b>A1</b>
3.1 Technical data.....	17		
3.2 Product image/scope of delivery .....	18		
3.3 Product's identification label .....	18		

## 1 Introduction

Dear Customer,

You have chosen a high-quality toilet seat raiser from Bischoff & Bischoff.

Thank you for trusting in us.

This User manual contains all the necessary information for assembling, adjusting, using and looking after the height-adjustable TSE-A toilet seat raiser.

Before using your new toilet seat raiser for the first time, please read and follow the information in this user manual carefully,

including all safety information. This manual is an integral and necessary part of the product. Keep the User manual within easy reach and pass it on with the toilet seat raiser when transferring the product.

For individuals who are visually impaired, this document is available as a pdf file on our website [www.bischoff-bischoff.com](http://www.bischoff-bischoff.com).

### IMPORTANT!

Despite thorough research and editing, incorrect information may occur in our user manual. We reserve the right to make technical modifications. Figures may show accessories. You can find the most current version of the user manuals in the download area of our website:

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### 2.1 Labels and symbols

#### IMPORTANT!

Indicates particularly useful information in a particular context.

You will find the explanations of any additional symbols used in the appendix at the end of this user manual.

### 2.2 Intended use

A toilet seat raiser supports an independent toilet seat usage caused by physical restrictions.

The height of the seat raiser needs to be correctly adjusted to the user.

### 2.3 Indications

Use of the toilet seat raiser is advisable for individuals with:

- poor body control,
- significant functional disorders in the lower extremities,
- stiffness in the hips and spine,
- handless persons, quadriplegics or individuals with arm disorders with significant disruption to the gripping function.

### 2.4 Contraindications

Use of the toilet seat raiser is not advisable for individuals with:

- Perceptual disorders,
- Balance disorders,
- No capability to sit


## 2.5 General safety information

Notification of incidents: The provider or user must report

1. any malfunction,
  2. any change in the characteristics or performance as well as
  3. any inadequacy in the labelling or instructions for use of a medical product, which has led or could lead to death or a severe deterioration in
- the health condition of a patient, an employee or a third party, must be reported immediately to the national authority responsible for medical products.

## 2.6 Special safety information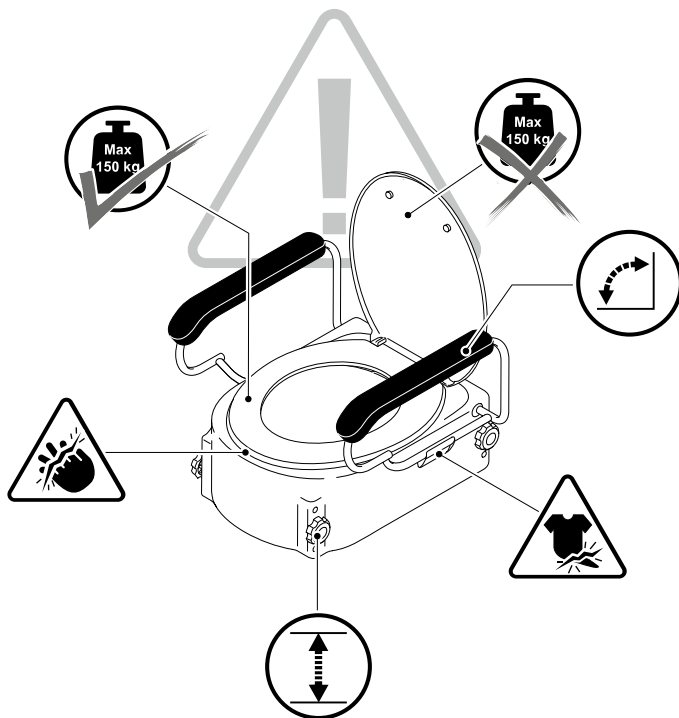

### IMPORTANT!

You will find the description of any symbols used in the appendix at the end of this User manual.

## 2.7 Service life

The expected service life is up to 5 years, if the device is used correctly, and safely, following the care and maintenance instructions.

Usage beyond this period is at own risk and is only allowed if the product is in a condition allowing its safe usage.

## 2.8 Declaration of compliance

B+B as the product's manufacturer declares that the TSE-A toilet seat raiser conforms to the requirements of the 93/42/EEC Directive for medical products.

## 3 Components/Overview

### 3.1 Technical data

Total length:	485 mm
Total width:	575 mm
Height (with the toilet lid closed):	375 mm
Armrest height (from the toilet seat):	200 mm
Seat width:	360 mm
Seat depth:	390 mm
Front/rear seat height adjustment:	80 mm, 120 mm, 160 mm (from the toilet bowl)
Hygiene cutout (funnel shaped):	245 x 270 mm (front) 230 x 210 mm (at the back)
Unladen weight:	4.9 kg
Max. payload:	150 kg
Colour:	white
Frame material:	Polypropylene (PP)
Armrest material:	PU foam

### 3.2 Product image/scope of delivery

On receiving the goods, please check that all components have been delivered. The package should contain the following components (Fig. 1 and 2):

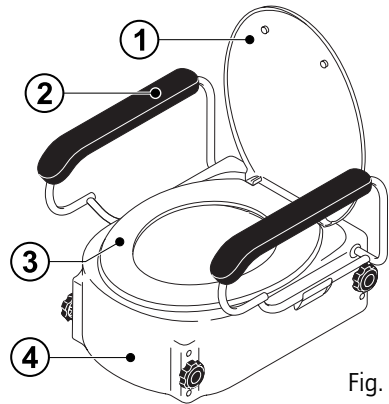
Fig. 1

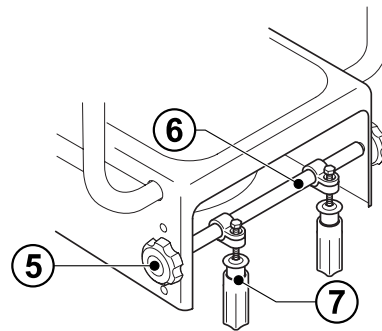
Fig. 2

- ① Toilet lid (1x)
- ② Padded armrests (2x)
- ① Toilet seat (1x)
- ④ Seat base (1x)

- ⑤ Hand screw (4x)
- ⑥ Tubes (2x)
- ⑦ Adjusting screws (2x)
- User manual (1x)
- Packing

### 3.3 Product's identification label

The product's identification label (Fig. 3) is located on the rear tube ⑥.

- Ⓐ Manufacturer's logo
- Ⓑ Manufacturer information
- Ⓒ Date of manufacture
- Ⓓ Model name
- Ⓔ Catalogue number
- Ⓕ Medical product in compliance with EU Regulation 2017/745
- Ⓖ CE mark
- Ⓗ Read the User manual!
- Ⓘ Maximum payload
- Ⓙ Unique Device Identification (UDI)
- Ⓚ Serial number





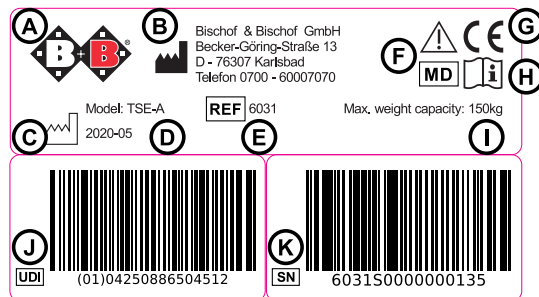
Fig. 3

### 4 Armrests assembly and/or adjustment

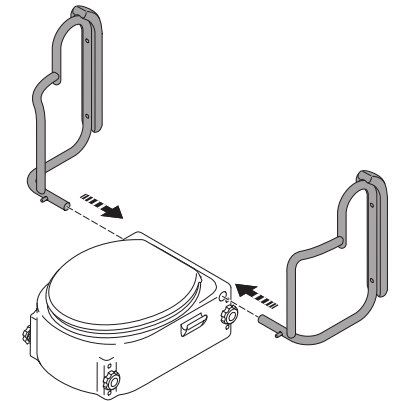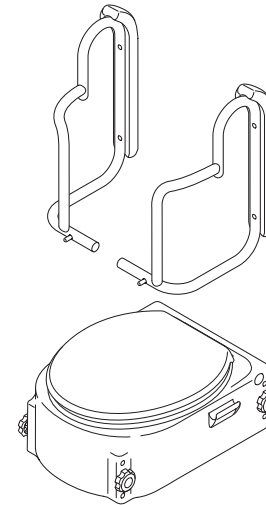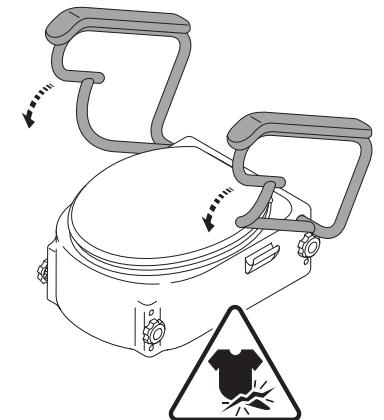
1 2  
3 4

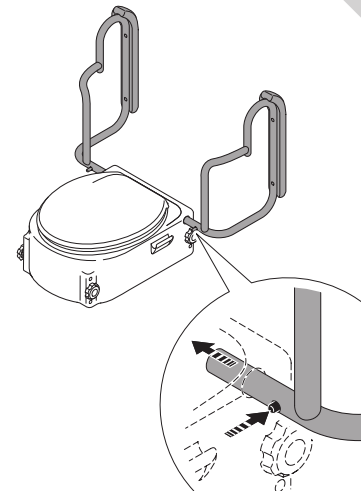
#### IMPORTANT!

Remove the armrests by following the instructions in reverse order.


### 5 Adjusting the seat height

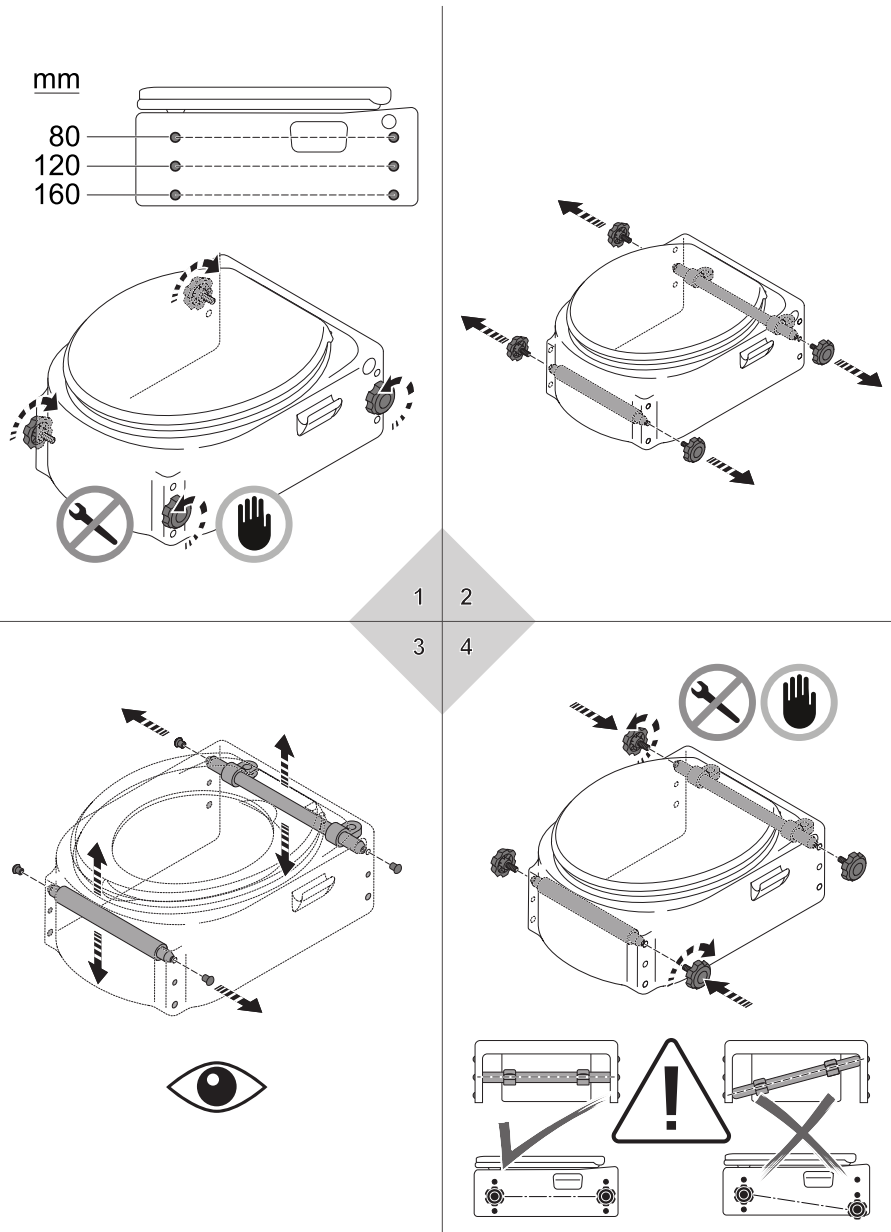### 6 Mounting the toilet seat raiser
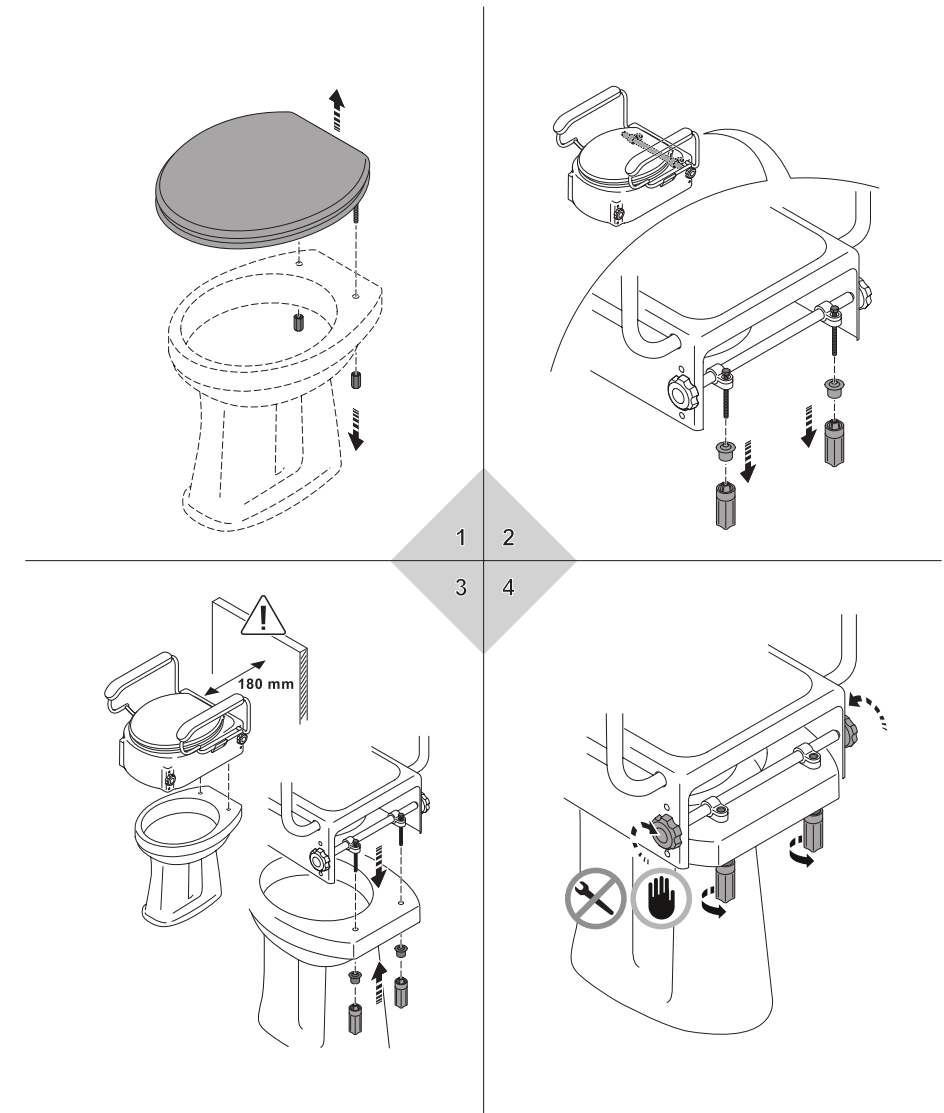
## 7 Usage and care
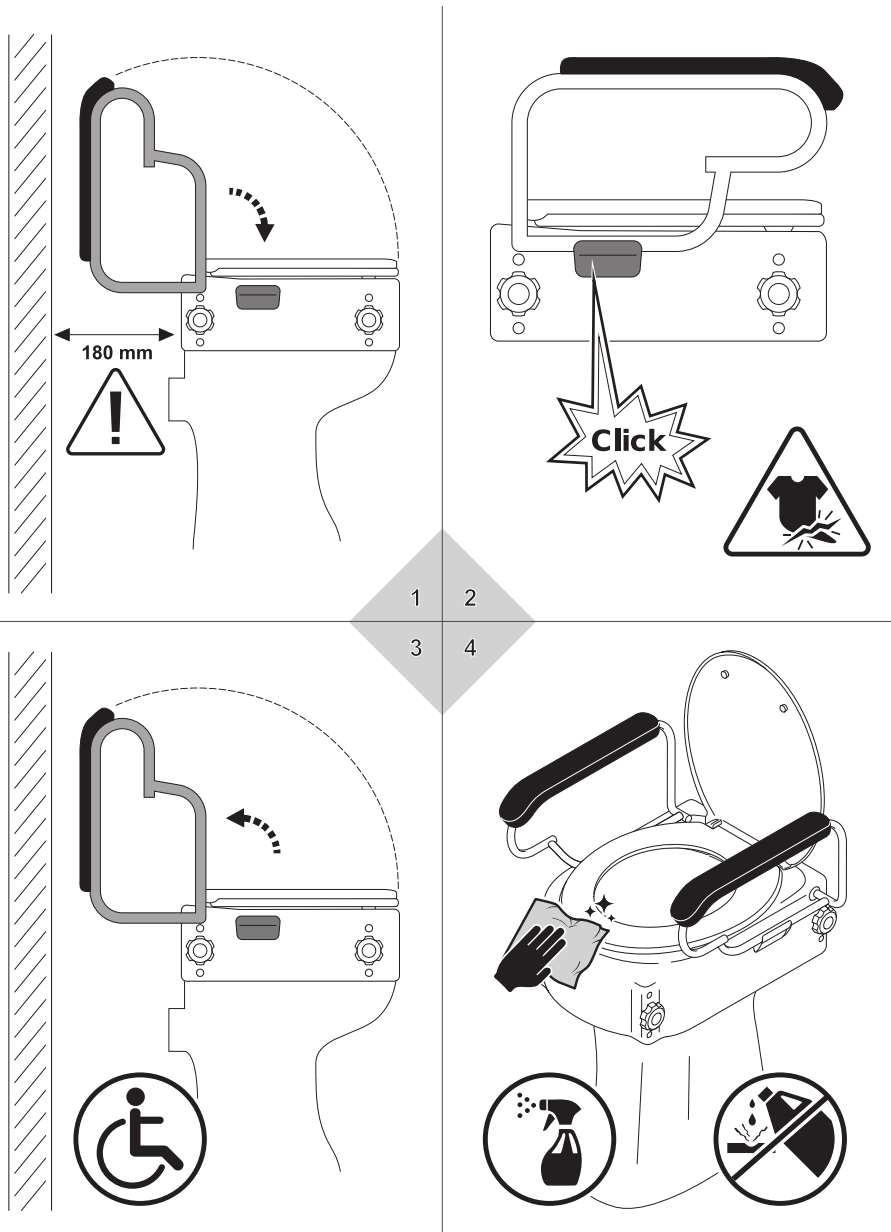
## 8 Maintenance and repair

Your toilet seat raiser is made from high-quality materials for long-term, comfortable use. We carry out in-depth testing of all our products before they leave our factory. However if you should find that your toilet seat raiser has a fault, you should no longer use it. Contact the healthcare professional who supplied the product to you to have the product repaired. They will be happy to help you.

Furthermore, we recommend that the toilet seat raiser is serviced at least once a year.

Your dealer will receive a maintenance plan from us together with further documents which are necessary to carry out an inspection.

Only authorised partners (medical supply stores or dealers) are permitted to carry out repairs and maintenance work.

If you have any questions about maintenance and repair, our Customer Service will be happy to help on the free phone number +49 (0)700 6000 7070.

## 9 Transfer/Re-use

Your toilet seat raiser is designed for re-use. Before passing the toilet seat raiser to another user, it needs to be serviced and disinfected.

When the toilet seat raiser is passed to another user, please also hand over this user manual and the address of your dealer.

## 10 Storage and shipping

Before you store or ship the product, remove all parts which have been plugged in and are not fixed parts. First, pack these individually and then pack them all together in a suitable cardboard box.

Where possible, retain and use the original packaging for this purpose. In this way the product has the highest level of protection against environmental influences during storage or transportation.

## 11 Disposal

Please contact your dealer for professional disposal of the product.

If you would like to dispose of the product yourself, please observe the local municipal regulations!

## 12 Warranty

The warranty covers all defects which are proven to be due to faulty material or manufacturing. In the event of a claim, the fully completed warranty certificate must be submitted with the product. B+B provides the legal warranty of 2 years.

User manual as well as not carrying out maintenance work properly and/or making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

Failure to observe the instructions in the

### 12.1 Warranty certificate

<b>Product:</b>	<b>TSE-A toilet seat raiser</b>
Model number:	
Serial number: (*)	
(*) To be filled in by the dealer. The product's identification label is located on the rear tube	
<b>Dealer:</b>	
Date and stamp	
(Pay attention to our general terms and conditions.)	

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**B+B France S.A.R.L.**