Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi


DE

EN

ES

FR

Enjoy mobility.

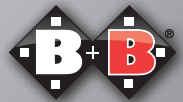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Dear customer,
 This instruction manual is designed to familiarise you with the use and maintenance of your B+B walking aid and to assist in making your life safer. We hope you enjoy our product and are more mobile with our product.

1. Unfolding

Swivel sides outwards by handles until you hear them lock.

2. Folding

Press the telescopic lock and at the same time fold first the right then the left side inwards.

3. Adjusting frame height

Press spring pin and allow the tube to click into the orthopaedically correct position.

4. Caution - safety instructions - Risk analysis as per DIN EN ISO 13485

- Max. load capacity: 120 kg
- Beware of crushing hands while folding the frame.

Instructions for adjusting the height:

- Make sure that the spring pins are correctly seated.
- Set all four feet to the same height.
- Place on solid, horizontal surface only.

5. Technical data

	Rezi	Gebo
Weight	2,5 kg	2,2 kg
Height	82 - 92 cm	82 - 92 cm
Braces	3	2
Adjustment	5	5
Colour	Bronze	Bronze
Width (folded)	53 cm	58 cm
Depth	10 cm	10 cm
Height	variabel	variabel

Warranty certificate

Guarantees refer to all faults of the product that are attributed to material or manufacturing faults. In the event of complaints, the complete, filled in warranty certificate shall be submitted.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of B+B will generally lead to forfeiture of warranty and product liability claims.

Our products come with a statutory guarantee of 2 years. For replacement devices and repairs, a warranty period of 1 year applies. Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

Warranty certificate

Product: Rezi / Gebo

Model number:

Serial number:*

* (to be entered by dealer)

Dealer:

Date and stamp

(Please note our general business terms & conditions)

*See the side frame for information stickers

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Bischoff & Bischoff GmbH
B+B France S.A.R.L.